



CLINICAL ASSISTANCE GUIDE

ASPIRE VOGUE CARBON FIBRE WALKER

Aspire Vogue Carbon Fibre Seat Walker



The Aspire Vogue Carbon Fibre Seat Walker is a four wheeled, lightweight walker with a cross folding frame, making it light and easy to use, store and transport. Its European ergonomic design offers personalised height adjustment that promotes correct posture and the small turning circle offers excellent manoeuvrability and stability in tight spaces.

Available in sizes - medium (black frame) and tall (red frame), the handles are height adjustable for a customised fit. It is equipped with a mesh seat, supportive backrest and weatherproof shopping bag allowing the user to rest or carry small items.

One of the lightest seat walkers on the market, the Aspire Vogue Carbon Fibre Seat Walker epitomises function and style with a super lightweight carbon fibre frame for a superior walking experience.

FEATURES

- Super lightweight Carbon fibre frame to maximise portability
- Luxury look and feel Danish design, profiled tubing, moulded fittings, ergonomic handles and refined colours.
- Lightweight and ergonomic design with small turning circle for excellent manoeuvrability in tight spaces.
- AutoLocate Smooth height adjustment that automatically locates to pre-set heights, making adjustment simple and accurate.

- Simple folding mechanism locks for easy transportation and storage
- Advanced locking handbrakes for increased user safety when walking and sitting.
- Unbreakable TPE super soft wheels offers high durability and a smooth ride for increased comfort.
- Removable utility bag stylish and simple to remove for shopping. Can hold up to 5 kg.
- Safe Working Load 150kg; shopping bag 5 kg
- Now supplied with bonus cane holder

IDEAL USER CONDITIONS

The Aspire Carbon Fibre Seat Walker may assist those who require support with walking due to the following issues:

- · Joint pain in the back, hips, knees or feet due to arthritis, osteoporosis or gout
- Back pain or spinal canal stenosis
- Reduced balance due to a neuromuscular condition such as:
 - Multiple Sclerosis
 - Parkinson's disease
 - Motor Neurone Disease
 - Stroke
- Fatigue or need for frequent rest breaks
- Chronic Obstructive Pulmonary Disease and breathlessness



CLIENT OUTCOMES

Increased confidence and independence both inside and outside of the home:

- The lightweight, Aspire Vogue 4-wheel walker with its tight turning circle and good manoeuvrability, can support users to access areas they need to within their home
- The large wheels and lightweight frame make it easier to manage different floor surfaces and to lift up and down a threshold step
- The enhanced stability and support provided to those with impaired mobility, can increase the user's confidence and feelings of safety to continue their daily activities

Enhanced community access and social connection:

- The lightweight and easy folding frame makes the Aspire Vogue Carbon Fibre Seat Walker easy to lift into and out of a car, supporting access to the community and continued social engagement
- The seat and back support enable rest breaks when out and about, to assist pacing and energy conservation and reduce pain and fatigue. This can increase confidence in going out as the user can be assured that they can rest when needed

SPECIFICATIONS

Code	Overall Height	Overall Length	Overall Width	Overall Depth	Seat Width	Seat Height	Folded Width	Wheel Size	Min. Functional Weight	Typical Functional Weight	Bag SWL	SWL
WAF705350 Black - Medium	695-920 mm	680mm	600mm	240mm	460mm	550mm	225mm	8"	4.95kg	6.2kg	5kg	150kg
WAF705360 Candy Red - Tall	770- 1040mm	680mm	605mm	240mm	460mm	620mm	225mm	8"	5.2kg	6.35kg	5kg	150kg

WARRANTY

1 year

COMPLIANCE

• ISO 11199-2:2005

USER MANUAL

 Aspire Vogue User Manual: https://www.aidacare.com.au/globalassets/ knowledge/resources-page/aspire/aspire/ vogue walker user manual.pdf

MAINTENANCE

GENERAL:

- Clean the walker regularly with a damp cloth.
- Use a mild washing agent and warm water for excess dirt build up.
- DO NOT use harsh abrasives or bleach-based agents which may cause damage or discolouration.
- Dry with a soft cloth.

WHEELS:

- · Clean wheels with a mild washing agent and warm water. Remove hair and other dirt from axles.
- Lubricate with multi-purpose oil periodically or if wheels begin to squeak.

FRAME:

- Ensure that every part of the walker is secure at ALL times.
- Check all screws for tightness. If not, tighten if necessary.
- Replace any broken, damaged or worn items immediately



MAINTENANCE SCHEDULE:

Weekly	Periodically	Yearly	Serviceable Components
 Check brake function Check all wheels are secure 	 Check handgrips for wear Check frame bolts are secure Check folding hinge bolts are secure Check front and rear axles bolts are secure Check for wear or cracks Check wheels for splitting or damage Check seat upholstery for wear and seat base for cracks and that it is secured to frame 	Take your walker to an authorised reparier for full check, clean, service and lubrication of all moving parts	 Wheels Bags Hand Grips Forearm Supports Brake Cables Seat Brake Handles Bearings Nuts and Bolts

SUPPORTING EVIDENCE

Å. BRANDT, S. IWARSSON & A. STÅHL (2003) Satisfaction with rollators among community-living users: a followup study, Disability and Rehabilitation, 25:7, 343-353 https://www.tandfonline.com/doi/pdf/10.1080/0963828021000058495?

LINKS

No links avaliable

AIDACARE KNOWLEDGE HUB

Aidacare Website Link:

Aspire Vogue Carbon Fibre Seat Walker: https://www.aidacare.com.au/products/walking-aids/seat-walkers-rollators/aspire-vogue-carbon-fibre-seatwalker-rollator/

PDF Library:

- Aspire Voque User Manual:
 - https://www.aidacare.com.au/globalassets/knowledge/resources-page/aspire/aspire_vogue_walker_user_ manual.pdf
- Aspire Vogue Walker Brochure
 - https://www.aidacare.com.au/globalassets/knowledge/resources-page/aspire/vogue_walker_brochure.pdf
- Aidacare Choice Assistance Guide Walkers
 - https://www.aidacare.com.au/globalassets/knowledge/resources-page/cat-service-user-guides/walkers_choice_ assistance_quide.pdf
- Vogue Carbon Fibre Walker Flyer
 - https://www.aidacare.com.au/globalassets/knowledge/resources-page/aspire/vogue-carbon-fibre-walker-tall-
- Safe Use Guides Walking Frames & Rollators:
 - https://www.aidacare.com.au/globalassets/knowledge/resources-page/cat-service-user-guides/guides_for_ installation_-_walking_frames_rollators.pdf

Video Library:

Aspire Vogue Walker Range:

https://www.youtube.com/watch?v=y1LorgrStSE&ab_channel=Aidacare

