

BATH TRANSFER BENCH WITH ROTATING SEAT

USER MANUAL



Congratulations and thank you for choosing the Aspire Bath Transfer Bench with rotating seat.

This user manual contains a description of the product and important guidelines to ensure correct and safe use. It is important to read this manual carefully prior to use.

It is especially important to read the safety requirements and follow these.

Aidacare continuously improves products and reserves the right to change the specifications and functions of products without notice.

If you have any queries please contact your dealer or Aidacare directly.

© Aidacare Limited 2024

No part of this user manual may be copied, distributed, reproduced, scanned or stored on any electronic database, whether in whole or in part in any form or by any means.

Due care has been taken to ensure all the information contained in this user manual is correct at the time of printing. All measurements, pictures, colours and weight capacities are to be used as a guide only. We reserve the right to modify the design or appearance of any product displayed in this user manual without prior notice.

TABLE OF CONTENTS

1.	Product Description & Intended Use	4
2.	Symbols Used in This Manual	4
3.	Symbols Used on the Product	5
4.	Warnings	5
5.	Product Specifications	8
6.	List of Components	9
7.	Assembly Instructions	10
8.	Instructions for Use	11
9.	Care, Maintenance & Customisation	13
10.	Warranty & Compliance	13
11.	Manufacturer	14

1. PRODUCT DESCRIPTION & INTENDED USE

The Aspire Bath Transfer Bench with rotating seat is designed to assist individuals with limited mobility in safely transferring in and out of the bathtub. It features a lightweight aluminium frame, rotating seat for easy rotation, height-adjustable legs to accommodate different bathtub heights and user preferences, a non-slip surface for stability, and backrest and armrests for additional support and comfort. To maximise safety, make sure to always check the bench for stability before use and do not exceed the weight capacity specified.

2. SYMBOLS USED IN THIS MANUAL



WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage or malfunction. On the product, this icon is represented as a black symbol on a white triangle with a black border.



ALWAYS! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white infinity symbol on a black dot with a white border.




DO NOT! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol with a black circle and black slash.

3. SYMBOLS USED ON THE PRODUCT



Please note: Stickers are not shown to scale.

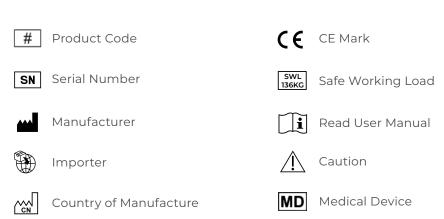



4. WARNINGS



WARNING! This user manual must be read before using the Transfer Bench. The manufacturer does not take any responsibility for damage and/or injuries caused by the fact that the User Manual has not been followed.



WARNING! The Transfer Bench must not be used without supervision or assistance by users with serious difficulty balancing or whose muscle strength is too low to stand or sit.



WARNING! The decision to use a Transfer Bench should always be guided by a qualified health care professional, who can also advise and assist with the correct height adjustment and usage instruction.



WARNING! Make sure all four legs are safely and securely on the ground before use.



WARNING! The suction cups legs should be placed on the Support Handle side of the Bath Transfer Bench.



WARNING! The spring buttons must protrude fully through the hole after the leg has been adjusted.



WARNING! Seek advice from a qualified health professional for the appropriate height level to suit your needs.



WARNING! You should always use the safety lap belt when sliding or rotating the seat. It should only be removed when transferring off the seat.



WARNING! Always test the stability of the transfer bench before use.



WARNING! Assistance when using the transfer bench is recommended for users with limited mobility.



WARNING! Use a slip proof bath mat to increase safety when using the transfer bench.

LIMITATIONS FOR USE

SAFE WORKING LOAD (SWL)





DO NOT exceed the safe working load.



DO NOT sit on transfer bench whilst attempting to adjust seat height.



ALWAYS ensure the height adjustable legs are locked into place before use.



ALWAYS ensure the height adjustable legs are levelled after placing in the bath before use.



ALWAYS use the transfer bench only for the purpose it is intended for.



DO NOT use the bath transfer bench without adjusting the height adjustable legs to the user's height.



DO NOT use the bath bench without the backrest in place.



DO NOT sit on the bath bench straddling the support bars.



DO NOT rotate the bath bench across the support handles.



ALWAYS ensure that the bath bench is stable when placed into the bath.



ALWAYS place the side of the Bath Transfer Bench with the support handle and suction cup legs in the bath and not the other way around.



ALWAYS ensure that the support handle is placed into the bath to assist with the transfer into the bath.



ALWAYS ensure that the user is comfortably seated on the bath bench with their feet flat on the floor.



DO NOT modify on the transfer bench.



DO NOT attempt to replace any components or your transfer bench. If they are faulty or require replacement, please contact an authorised repairer.



DO NOT stand on the transfer bench.



ALWAYS ensure all legs ARE IN CONTACT with the ground at ALL TIMES during use.



NOTE: Fingers or other body parts can be trapped between the transfer bench's parts during use. Be extremely careful when using the item and observe the dimensions of gaps between the transfer bench's elements to avoid trapping your fingers or other body parts.

5. PRODUCT SPECIFICATIONS

Model	Overall	Overall	Overall	Seat	Seat	Seat	Product
	Height	Length	Width	Width	Height	Depth	Weight
	mm/in	mm/in	mm/in	mm/in	_{mm/in}	mm/in	kg/lb
Transfer Bench	793 - 905 / 31.2 - 35.6	474 / 18.6	951 / 37.4	424 / 16.6	503 - 617 / 19.8 - 24.2	346 / 13.6	8.1 / 17.8

6. LIST OF COMPONENTS

The Aspire Bath Transfer Bench with Rotating Seat is supplied in a box with all necessary components however some assembly will be required. The contents of the box include the main frame section, the backrest, and the four height adjustable legs.

BOX CONTAINS

- 1 Main frame
- 2 Backrest
- 3 2 x Height adjustable legs
- 4 2 x Height adjustable suction cup legs





1	Backrest	3	Lap belt	5	Support handle		Height adjustable
2	Rotating seat		Armrest	6	Height adjustable legs	7	suction cup legs



7. ASSEMBLY INSTRUCTIONS

 Make sure all carton components are included (Refer to page 9 for full contents)



 Attach the backrest by lining up the backrest frame with the holes on the main frame and inserting it. Make sure the pins slot into their holes correctly as shown in images.





3. To attach the legs, line up the leg with the main frame section and insert onto the frame. The suction cup legs should be on the side with the support handle and the standard legs should be on the other side. A notable 'click' will be heard when the quick release button is engaged. Make sure to adjust the legs to the correct desired height prior to use. All legs should be at the SAME level.



8. INSTRUCTIONS FOR USE

INITIAL CHECK

Remove the transfer bench from the box and check the contents to ensure nothing is missing. If you discover any component is missing or damaged do not continue to assemble the product. Contact Aidacare immediately.

8.1 POSITION THE BENCH

Place the bench so that the side of the bath trasnfer bench with the support handle and the suction cup legs are inside the bath and the other side is outside. Make sure the bench is stable and the legs are securely on the ground.

8.2 ADJUST THE HEIGHT

The bench has height adjustable legs with 12.5 mm increments on all legs. Adjust the height so that it's comfortable for the user.

8.3 UNDERSTAND THE SEAT

The moulded seat comes with a safety belt and slides from end to end, allowing easy patient transfer. The seat also rotates, allowing easier access from any angle outside the tub.

8.4 LOCK THE SEAT

The seat has a 360° rotation and can be locked at 90° intervals. Make sure to lock the seat in a comfortable position for the user.

8.5 TRANSFERRING ONTO THE BENCH

If transferring from a wheelchair, place the wheelchair next to the bench, facing the front of the transfer bench. Transfer your torso to the transfer bench first, then lift your legs into the tub. Be sure to turn slowly, without twisting, to prevent injuries.

8.6 UNSAFE USE OF YOUR BATH TRANSFER BENCH



DO NOT
rotate the bath
seat in the direction of the
support handle.



DO NOT reach or lean while sitting on the bath seat.



DO NOT use the bath transfer bench without adjusting the height adjustable legs to the user's height.



DO NOT adjust the transfer bench while seated.



DO NOT sit on the transfer bench straddling the support bars.



use the transfer bench without the backrest in place.

9. CARE, MAINTENANCE & CUSTOMISATION

CLEANING INSTRUCTIONS

- · Wipe the transfer bench dry after every use.
- · Store away from direct sun or radiant heat.
- · Use a mild washing agent and warm water for excess dirt build up.
- DO NOT use harsh abrasives or bleach based agents which may cause damage or discolouration.
- · Dry with a soft cloth.

MAINTENANCE

- Ensure that every part of the transfer bench is secure at ALL times.
- · Check all screws for tightness. If not, tighten if necessary.
- · Replace any broken, damaged or worn items immediately.

CUSTOMISATION

All customisation, modification and or alterations made to this product are considered as 'custom' changes. They may cause the product to no longer meet safety standards and should be considered prior to commencement.

Any custom modification of this product is not warranted by Aidacare and will void the warranty.

10. WARRANTY & COMPLIANCE

Aidacare products are manufactured and independently tested to the market-relevant compliance standards.

Labels on the product address compliance requirements and users should familiarise themselves with these certificates.





11. MANUFACTURER

If you have any questions, you can contact:



Manufacturer:

Aidacare Pty Ltd
ABN 40 134 398 833
Building 3A, 1 Moorebank Avenue, Moorebank NSW 2170
1300 133 120 | product@aidacare.com.au

AIDACARE PTY LTD

Building 3A, 1 Moorebank Avenue, Moorebank NSW 2170 Australia

1300 133 120 | aidacare.com.au

MADE IN PRC

