

## **COMFIMOTION FIXED COMPANION BED**

## **ASSEMBLY INSTRUCTIONS**

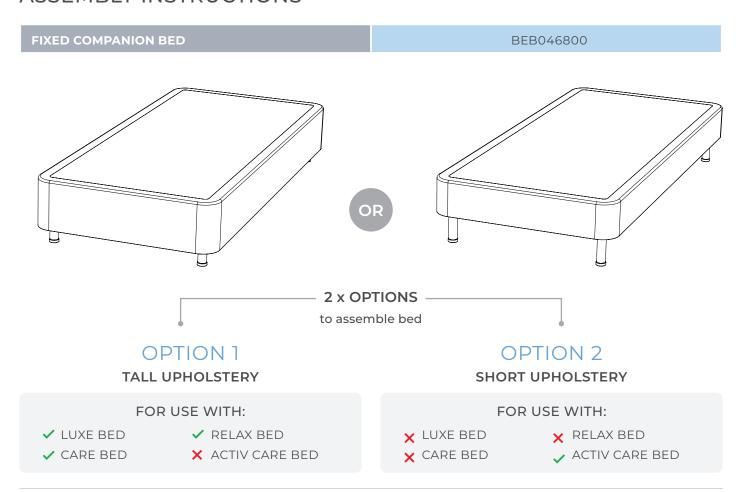

## **KIT CONTENTS:**


ITEM	QTY	DESCRIPTION
1	1	Frame
2	1	Upholstery
3	4	Fixed Height Legs
4	4	Leg Bolts
5	4	Tall Corner Pieces
6	4	Short Corner Pieces
7	16	Plastic Clips

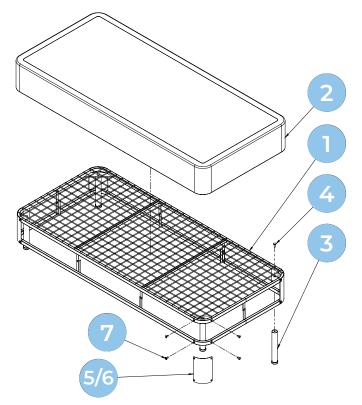




## **HOW TO ASSEMBLE:**

- 1 Cut the zip ties holding legs, corners, upholstery, hardware and packaging to frame and put to one side.
- 2 Flip frame 1 upside-down so that the mesh deck is lying on floor.
- Assemble the legs 3 to each corner of the frame. To do this, insert the bolts 4 through the square hole in the frame 1 and screw the legs to the bolts. Hand tighten only.

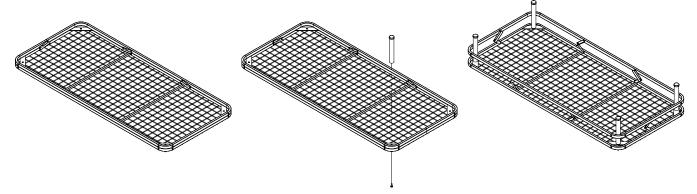


Gently pull the frame apart and install desired corner pieces on all four corners using the push in clips. Use tall corner pieces for tall upholstery option or short corner pieces for short upholstery option.

NOTE: Images shown are for TALL upholstery installation.

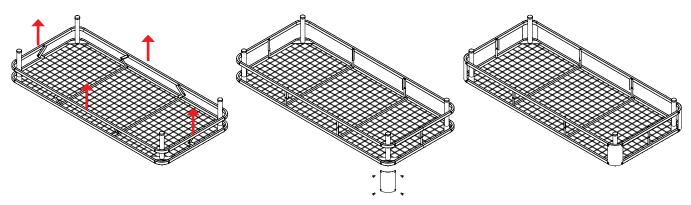

- 5 Flip the frame over so that its standing on its legs.
- 6 Slip the upholstery 2 onto the bed, fold excess underneath the frame and fix in place with the hook and loop strips sewn on internal side of upholstery.

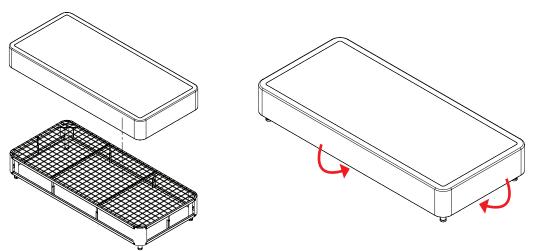

7 The bed is now complete!

