


MAXI BEDSIDE COMMODE

QUICK USE GUIDE



GENERAL INSTRUCTIONS:


1 Remove the seat cushion from the chair by lifting upwards.

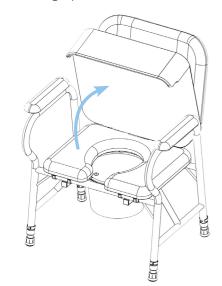

H E


2 Lift the seat and **remove** the lid from the bowl.

3 Ensure that the bowl is positioned securely under the seat when closing.

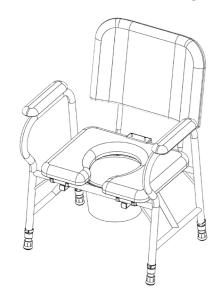

5 After use, empty and clean the bowl.

4 Before use, add a small amount of water to the bowl to facilitate easier cleaning later.

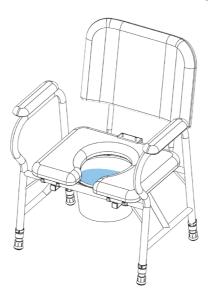

LEG ADJUSTMENT:

- To adjust the height of the commode, push the button in and slide the leg up or down to the appropriate hole.
- For safety, make sure the button is fully protruding from its designated hole after adjustment.

WIDTH ADJUSTMENT:

- The commode has a fully adjustable width to accommodate different seat sizes and improve comfort.
- \cdot $% \left(Adjust the width based on the seat size to remove obstructions and \right)$ enhance user comfort.


SAFETY PRECAUTIONS:

- Ensure all four legs of the commode remain in contact with the ground during use.
- · Check the stability of the commode on uneven surfaces and adjust individual legs as needed for stability.

MAINTENANCE:

- Regularly clean and sanitize the commode to maintain hygiene.
- Inspect the commode for any signs of wear or damage and replace any worn-out parts promptly.



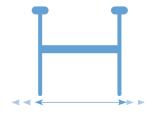





AIDACARE PTY LTD

Building 3A, 1 Moorebank Avenue, Moorebank NSW 2170 Australia

1300 133 120 | aidacare.com.au

AIDACARE LIMITED

Arcadia House, Maritime Walk, Ocean Village, Southampton SO14 3TL United Kingdom

01622 541235 | aidacare.co.uk

MADE IN PRC

ASUM109.4 270324

