



Panamatic Midi Bedpan Washer Disinfector

Redefining Infection Control





Redefining Infection Control

Our range of washer disinfectors have been developed over decades to provide you with a fast and safe human waste disposal system with passive safety measures to minimise the risk of infection. Designed to empty, wash and disinfect human waste containers such as bedpans, commode pots and urine bottles, our bedpan washer disinfectors incorporate the latest technological advances.

The Panamatic Midi provides clinicians with a medium capacity bedpan washer with a wide-opening lid for ease of loading.

Capable of disinfecting up to 3 reusable items at a time in a 7-minute cycle, the Midi is ideal for standard recovery wards and small care settings.

Benefits

1

Reduce cross contamination of infectious materials and minimise the risk of ward outbreaks, keeping staff and patients safe.

2

Incorporates the latest technological advances and able to achieve thermal disinfection levels in line with NHS periodic testing guidelines.

3

Medium capacity model to suit a variety of healthcare environments.

4

Extremely thorough wash performance, eliminating 99.99% of known bacteria.




1300 133 120




www.aidacare.com.au


Panomatic Midi Features


4 **Mid-range capacity to suit high usage environments**
– can clean up to 2 bedpans and 2 urinals per cycle.

 **Rapid 7-minute washing disinfection cycle**
– eco-friendly and cost effective.

 **Completely hands-free, no touch operation**
– reducing the risk of infections spreading to clinicians and further patients.

 **Top-loading**
– for a simple loading and emptying process.

 **High quality materials and components**
– including antimicrobial coatings and smooth, crevice-free surfaces, reducing the presence of trapped harmful bacteria.

 **Antimicrobial surfaces**
– to minimise cross infection and reduce the possibility of microbial growth.





Automatic scale management system

– suitable for soft and hard water areas, to prevent blockages.



Unique multi-utensil racking design

– extremely convenient with one rack to fit all standard bedpan sizes.



Rotating wash nozzles and strategically sited spray jets

– ensure a superior wash performance.



Optional programmable audio

– in your own language to ensure correct and proper use by staff.



Compact – to save space, making it suitable for all room types and sizes.



Standard electrical supply –

so you can connect and go!



1300 133 120



www.aidacare.com.au

Technical Specifications

General

Stainless Steel grade:	304
Dimensions w x h x d (lid closed):	500 x 1042 ±3 x 600mm
Dimensions w x h x d (lid open):	500 x 1500 ±3 x 600mm
Weight (Machine Only):	71kg
Weight (Packed):	86kg
Weight (Export):	91kg
Capacity:	2 pots + 2 urine bottles
Operation cycle time:	7 minutes
Decibel rating:	47.4dBA

Electrical

110V Voltage:	N/A
230V Voltage:	230V / 50Hz with dedicated 13A breaker (Minimum)
230 Voltage:	230V / 60Hz with dedicated 13A breaker (Minimum)
Circuit protection (230 Voltage):	30mA RCD, GFI, IEC60364 compliant
Cable connection (230 Voltage):	3 core sheathed x 1.5mm ² . 2m long
3 Phase Voltage:	N/A
Circuit protection (3 Phase Voltage):	N/A
Cable connection (3 Phase Voltage):	N/A
Rated power input:	2.55kW
Electrical consumption per cycle:	0.275kWh
Steam generator power:	2.5kW
Motor power:	N/A
Pump power:	0.425kW

Water


Water connection:	¾" BSP male fittings required
Water pressure:	Cold water 0.5-10 bar
Water flow rate (Min):	10L per minute at 1 bar
Water tank capacity:	20L
Water consumption per cycle:	21.2L
Waste connection:	75mm ID (3") Pushfit
Overflow connection:	22mm

Disinfection

Recommended disinfection detergents:	EcoWash
Disinfection tank volume (external):	5L
Disinfection dosing per cycle:	30mL

*Specifications subject to change

Dimensions

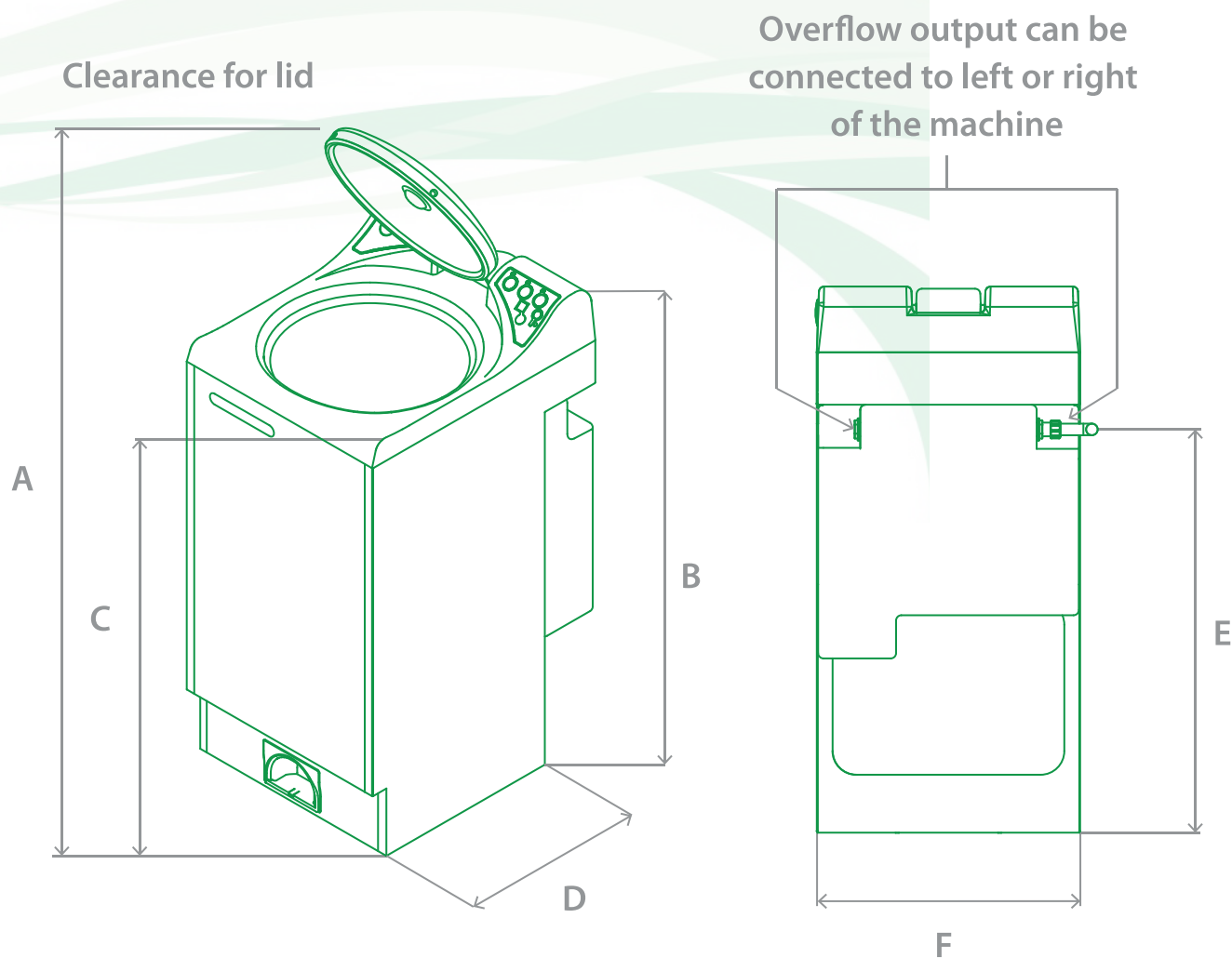

	Millimetres
A	1500 ±3
B	1042 ±3
C	960
D	600
E	766
F	500





DDC Dolphin is committed to providing you with the sluice/dirty utility room solution which best meets your needs. Get in touch to discuss your requirements.

Accreditations



Certificate No. 1161
ISO 9001



DDC Washer Sanitisers meet or exceed all the essential performance & design elements of ISO 15883 and AS 4187 (2019) as tested and certified by Australian Certification Services. Efficacy tested & certified by Eurofins AMS Laboratories.



Exclusively Distributed by:

1300 133 120
www.aidacare.com.au

